




SCAN THE QR CODE TO WATCH THIS 2 MINUTE HOW TO VIDEO BEFORE FIRST USE.

LIVER FLUKE RAPID ANTIBODY TEST for SHEEP



For use by Vets, Farmers and Livestock Handlers

KIT COMPONENTS

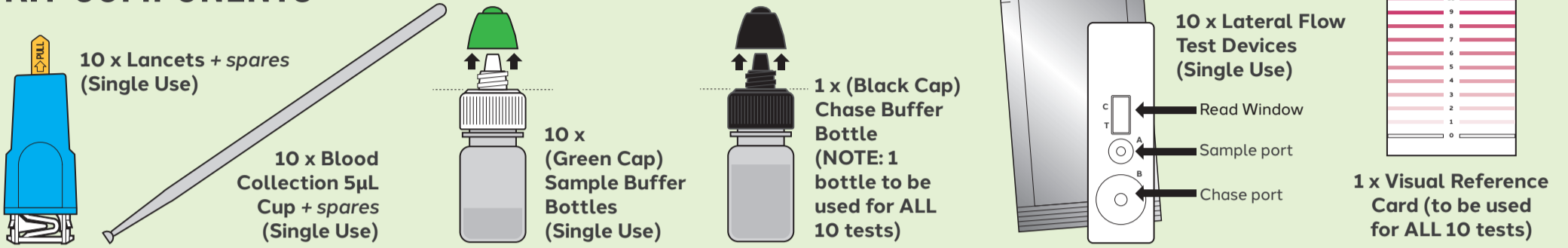

- Each test is a single use test. Dispose after use. Each lateral flow device can only be used once.
- Use within one hour of opening.
- Do not use after the expiry date.
- Store at room temperature (15° - 25°C).
- The test is designed for use with blood, but can be used with plasma or serum.

SPECIMEN COLLECTION AND HANDLING

1 Collect blood from inside ear (preferred sample site) or on top of the nose (where hair meets the nose).
If using ear, rub the inside of the ear for a few seconds to increase blood flow. If using the nose, dry the area first with a clean tissue.

2 Take lancet from the kit, and pull out the yellow tab to remove.

3 Apply lancet to sampling site. To activate the lancet, press the white button (1). Lancet blade will pop in and out. Lancets are single use only.
Wait 15 secs for the blood droplet to appear (2) before taking sample.
Gently collect blood sample to fill the collection cup (3).

4 Remove the whole sample buffer bottle lid (green/white), insert blood collection sample and swirl.
Replace lid, then gently invert to mix 10 times.
Use within 30 minutes.

5 Remove the lateral flow device from the foil pouch and ensure that the desiccant sachet is orange. (If green, discard and use new device).

6 Remove just the green cap.
Add 1 drop of sample blood buffer to the sample buffer port (A).
Replace green cap and close.

7 Within 0-2 minutes, remove just the black cap from the chase buffer bottle.
Add 3 drops of chase buffer on to chase port (B).
Replace black cap and wait 10 minutes before reading the test result.

INTERPRETATION OF TEST RESULTS

After 10 minutes evaluate the lateral flow device for the formation of a control line (C). If there is no visible control line the test is invalid.

If a control line (C) is visible, use the reference card to match up the colour of the test line (T) and assign a score to the test line. If there is no visible (T) line after 10 minutes, the animal has tested negative. If there any visible (T) line results, discuss with your vet.

Test score indications: 0-1 = Negative 2 = Inconclusive 3-10 = Positive

